
Fig. 2 / Rys. 2 / Рис. 2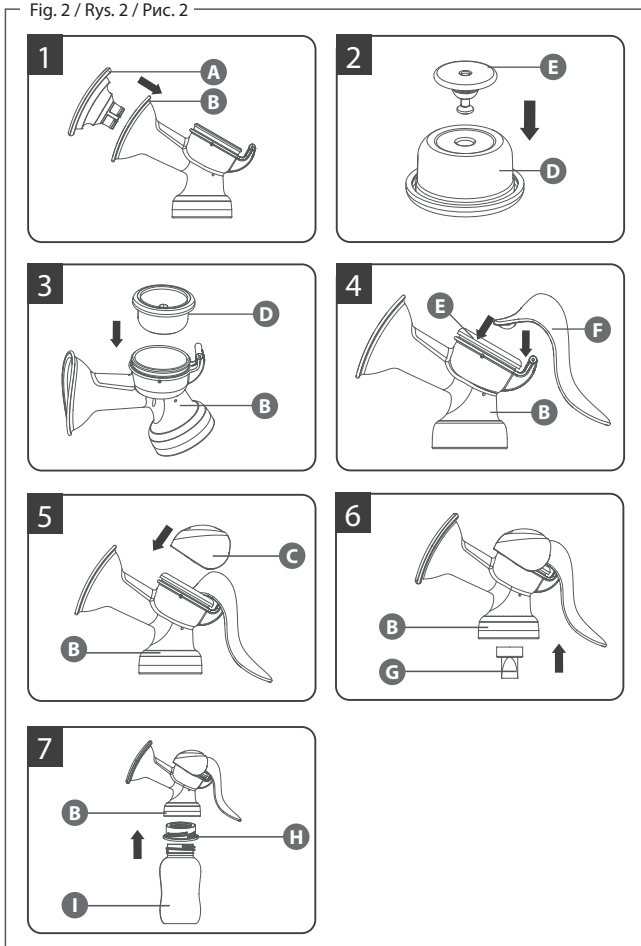

Instructions for Use of Manual Breast Pump

This manual breast pump is designed for expressing excess milk. It is not intended for expressing milk when a hungry baby cannot empty the breast themselves, i.e. in all cases when your milk ducts are blocked. Doctors recommend breast milk as the best food for babies in the first year of life, which should be combined with solid food from 4-6 months of age. Remember that your milk is the food that fully meets your baby's needs and it has antibodies that protect your baby from infections and allergies.

WHAT IS IN THE SET, fig 1

A – silicone massage cushion, B – breast pump body with a funnel, C – cover, D – silicone diaphragm, E – diaphragm and body connector, F – handle, G – silicone valve, H – bottle adapter for wide- and narrow-neck bottles, I – milk container 120 ml

HOW TO ASSEMBLE/DISASSEMBLE, fig. 2

To disassemble the breast pump, use the assembly instructions below following the steps in reverse order.

2.1 Place the silicone massage cushion (A) in the breast pump funnel (B). Press the narrower part of the massage cushion against the narrow neck of the funnel. Make sure that the edges of the cushion fit evenly to the edges of the funnel. Take note that both elements are profiled and should fit properly.

2.2 Holding the diaphragm (D) in one hand (concave side down), place the bolt (E) in the diaphragm's hole. In this way, the wider end of the connector (E) becomes the base of the diaphragm and the narrower end protrudes downwards.

2.3 Place the diaphragm with the bolt (D) in the cup of the breast pump body (B). Press down the edges of the diaphragm to fit it correctly. Make sure the membrane fits tightly to the edges of the cup.

2.4 Slide the handle (F) over the protruding part of the connector (E) to connect those parts. Then fit the handle onto the protruding hinge of the breast pump body (B).

2.5 Place the cover (C) on the breast pump body (B) and push until you hear a click.

2.6 Grasp the body (B) and place the silicone valve (G) by putting its round part inside the body's thread. Make sure it fits tightly to the edge of the hole.

2.7 Screw the adapter (H) onto the milk container (I). Then screw the set onto the breast pump body. The breast pump is now ready to use.

NOTE! Always remove the anti-return valve by grasping its round base with two fingers. Never grasp the tip of the valve, as it can be damaged and weaken the suction power. Wash the non-return valve very gently and do not allow food to dry inside it.

HOW TO CLEAN AND DISINFECT BREAST PUMP PARTS

Wash your hands thoroughly before expressing the milk. Disassemble the breast pump. Wash all parts of the breast pump in warm water with mild detergent. Then rinse them thoroughly and disinfect them according to the guidelines below. **Before first use**, disinfect the above-mentioned parts by placing them in boiling water for 5 minutes to ensure hygiene.

Before each subsequent use, disinfect the above-mentioned breast pump parts in a steam steriliser or by pouring boiling water over them to ensure hygiene. **NOTE!** After disinfection wait until all parts have cooled down. Do not start assembling the body when the parts are warm or moist as it can cause breast pump malfunction. Check that the individual parts of the breast pump are correctly attached.

HOW TO CLEAN AND DISINFECT BOTTLE/CONTAINER

Thoroughly wash and disinfect the milk container and its components before the first and each subsequent use. Place it in boiling water for 5 minutes to ensure hygiene. Clean before each use. Boiling and hard water may create a limescale effect. Wash all parts thoroughly with clean water, including areas that may not be visible but through which food has passed to remove any food residue. After use, clean the milk container of milk residues, rinse, wash and then disinfect in a steam/ electric steriliser or boiling water. The milk container can be used in a dishwasher (max. temp. 60°C). Do not use antibacterial agents for cleaning. Take special care when heating food in a microwave. Always stir heated food to ensure even distribution of temperature and check the temperature before feeding. If you heat food in a microwave, remember that the milk container must be open. The product should not be placed in a preheated oven or heated on a gas or electric stove.

For your child's health and safety. Warning! Inspect carefully before each use. Throw away at the first signs of damage or weakness. **Warning!** This product is not a toy. Keep away from reach of children when not in use. **ALWAYS** check the food temperature before feeding. Keep maximum hygiene when using the containers. **NEVER** boil expressed milk. **DO NOT** add fresh milk to defrosted milk. **DO NOT** refreeze previously defrosted milk. **NEVER** fill the container with hot liquids. Always use this product under adult supervision. For use with breast milk only. Not to be used for mixing formula milk. Retain the instruction for later use.

MILK STORAGE

Milk can be kept in a bottle or special food containers. Freeze only fresh milk. Do not fill a container fully as frozen milk increases its volume. Breast milk can be heated in the same way as any other food (but never in the microwave as this may destroy valuable nutrients and antibodies). Frozen breast milk can be defrosted overnight in the refrigerator and then warmed gradually in a bottle warmer or hot water. After defrosting, shake the container or stir the milk to mix the separated fat particles with the remaining volume of milk. Thawed milk can be used for 24h. It should be stored in a refrigerator. **NEVER** refreeze previously defrosted milk. **DO NOT** add fresh breast milk to defrosted milk. **DO NOT** add fresh breast milk to frozen milk. **NEVER** heat breast milk by boiling or putting it in the microwave. **ALWAYS** check the liquid temperature before feeding. Dispose of any milk that has not been used.

	Fresh breast milk	Thawed milk
Room temperature (+19°C to +25°C)	up to 4 h	do not store
Refrigerator (+4°C)	up to 96 h	up to 24h (since total thawing)
Freezer (-20°C)	3-6 months	never refreeze previously defrosted milk

The breast pump is intended for personal use. It is not refundable or exchangeable. It is covered by the manufacturer's warranty. Remember that the proof of purchase together with the stamped instructions for use are the basis to examine the complaint. **EACH BREAST PUMP MODEL IS TESTED IN TERMS OF USABILITY BEFORE IT IS RELEASED FOR SALE.** Remember that if you have lactation problems as a result of which your baby is unable to empty the breast, a breast pump may also be ineffective. **WARNING!** Do not use this product for purposes other than those listed in this user manual. Do not place the product near sources of fire. Before using the breast pump, make sure that all parts were assembled correctly. If you feel breast or nipple discomfort, stop using the breast pump immediately. Wash your hands before using the product. Keep the product out of the reach of newborns and children. Do not drop or hit the product to avoid damage. Do not touch product elements with bare hands immediately after disinfection in boiling water to avoid burns. Keep the instructions of use as it contains important information. The batch code indicated on the packaging.

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POHRANA MLJEKA

Milk can be kept in a bottle or special food containers. Freeze only fresh milk. Do not fill a container fully as frozen milk increases its volume. Breast milk can be heated in the same way as any other food (but never in the microwave as this may destroy valuable nutrients and antibodies). Frozen breast milk can be defrosted overnight in the refrigerator and then warmed gradually in a bottle warmer or hot water. After defrosting, shake the container or stir the milk to mix the separated fat particles with the remaining volume of milk. Thawed milk can be used for 24h. It should be stored in a refrigerator. NEVER refreeze previously defrosted milk. DO NOT add fresh breast milk to defrosted milk. DO NOT add fresh breast milk to frozen milk. NEVER heat breast milk by boiling or putting it in the microwave. ALWAYS check the liquid temperature before feeding. Dispose of any milk that has not been used.

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